

ESG-150


Controls and operation settings

Quick Reference Guide




Home screens

Advanced user level*

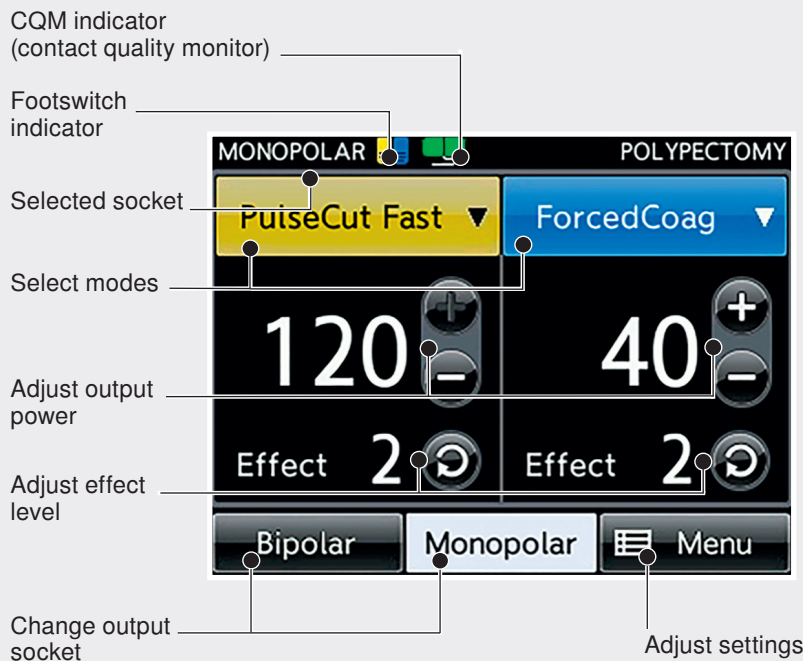


Basic user level*

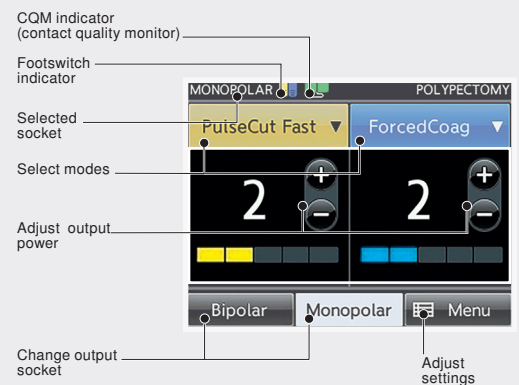

* The ESG-150 provides two different displays of the home screen, Advanced and Basic User Level. For detailed information refer to the instruction for use.

Monopolar modes

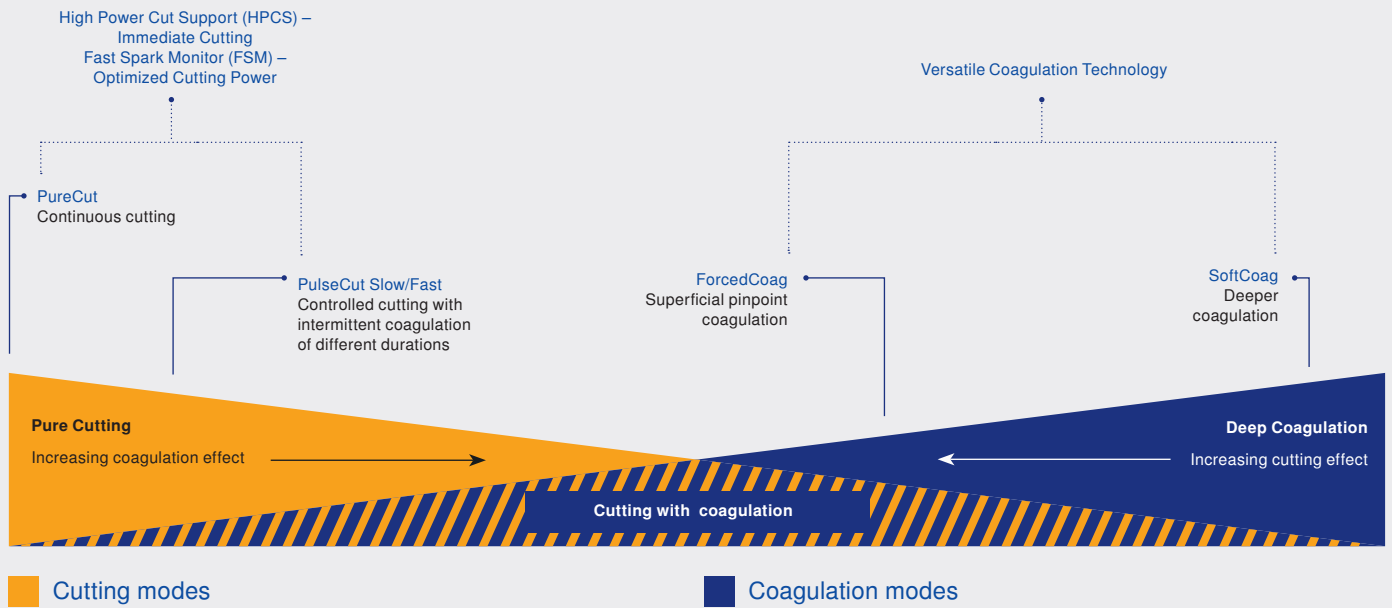

Operation settings

	Procedure	Instrument or technique	Mode	Advanced		Basic
				Power	Effect	Level
	Snare Polypectomy: Sessile polyp	Close the snare slowly and gently	PulseCut Slow	120 W	3	3
	Snare Polypectomy: Pedunculated polyp	Close the snare slowly and gently	ForcedCoag	40 W	3	4
	Endoscopic Mucosal Resection (EMR)	Close the snare slowly and gently	PulseCut Slow	120 W	3	3
	Hemostasis	Coagrasper	SoftCoag	50 W	4	3
	Sphincterotomy	Sphincterotome	PulseCut Slow	120 W	1	1