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PATIENT	Last name	Etternavn	
	First name	Fornavn	
	Middle name		
	Title	Suffix	
	Other patient name(s)		
	Other patient ID(s)		
	Ethnic group		
	Military rank		
	Patient ID	Personnummer **)	
	Date of birth	F.dato *) [dd.MM.yyyy]	
Sex	<input type="radio"/> Male <input type="radio"/> Female <input type="radio"/> Other		
Age	Years ▼		
Height	[m]		
Weight	[kg]		
Address			
Additional info			
<input type="button" value="Details"/>			
HOSPITAL	Referring physician	▼	
	Requesting physician	▼	
	Admitting diagnosis	▼	
	Ward	▼	
	Admission ID		
PROCEDURE	Accession No		
	Request ID		
	Requested procedure(s)		
	Study list		
	<input type="button" value="Add"/> <input type="button" value="Delete"/>		
	Study	▼	
	Patient position	▼	
	INSTITUTION	Institutions name	SSHF Arendal ▼
		1. Performing physician	▼
		2. Performing physician	▼
1. Operator		▼	
2. Operator		▼	
<input type="button" value="Preregister"/> <input type="button" value="Exam"/> <input type="button" value="Search"/> <input type="button" value="Cancel"/> <input type="button" value="Help"/>			

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<h3>PATIENT</h3> <p>Last name <input type="text" value="Etternavn"/></p> <p>First name <input type="text" value="Fornavn"/> [dd.MMM.yy]</p> <p>Patient ID <input type="text" value="Personnummer *)"/></p> <p>Date of birth D <input type="text" value="F-dato (**)"/> M <input type="text" value=""/> Y <input type="text" value=""/></p> <p>Age <input type="text" value=""/></p> <p>Sex <input type="radio"/> Male <input type="radio"/> Female <input type="radio"/> Other</p>	<h3>INSTITUTION</h3> <p>Ward <input type="text" value=""/></p> <p>Operator 1 <input type="text" value=""/></p> <p>Operator 2 <input type="text" value=""/></p>
<h3>STUDY</h3> <p>Accession No <input type="text" value=""/></p> <p>Study Description <input type="text" value=""/></p> <p>Study ID <input type="text" value=""/></p> <p>Study Comment <input type="text" value=""/></p>	
<p>Emergency Examination Preregister OK Cancel</p>	

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CT1 og CT2 SIEMENS CT

Velg *Patient* og *Register* ...

Patient	Applications	Transfer	Edit	View	Image	Tools	Scroll	Evaluation
Register...	Num 0							
Emergency...								
Browser...	Num .							

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PATIENT	Last name	<input type="text" value="Etternavn"/>	PROCEDURE	Accession No	<input type="text"/>
	First name	<input type="text" value="Fornavn"/>		Requested procedure(s)	<input type="text"/>
	Patient ID	<input type="text" value="Personnummer *)"/>		Study list	<input type="text"/>
	Date of birth	<input type="text" value="F-dato **) [dd-MM-yyyy]"/>		<input type="button" value="Add"/> <input type="button" value="Delete"/>	
	Sex	<input type="radio"/> Male <input type="radio"/> Female <input type="radio"/> Other		Study	<input type="text"/>
	Age	<input type="text"/> Years ▼			
HOSPITAL	Referring physician	<input type="text"/>	INSTITUTION	Institutions name	<input type="text" value="SSHF Arendal"/>
	Admitting diagnosis	<input type="text"/>		1 Performing physician	<input type="text"/>
	Ward	<input type="text"/>		1 Operator	<input type="text"/>
	Admission ID	<input type="text"/>			
<input type="button" value="Preregister"/> <input type="button" value="Exam"/> <input type="button" value="Search"/> <input type="button" value="Cancel"/>		<input type="button" value="Emergency"/> <input type="button" value="Help"/>			

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