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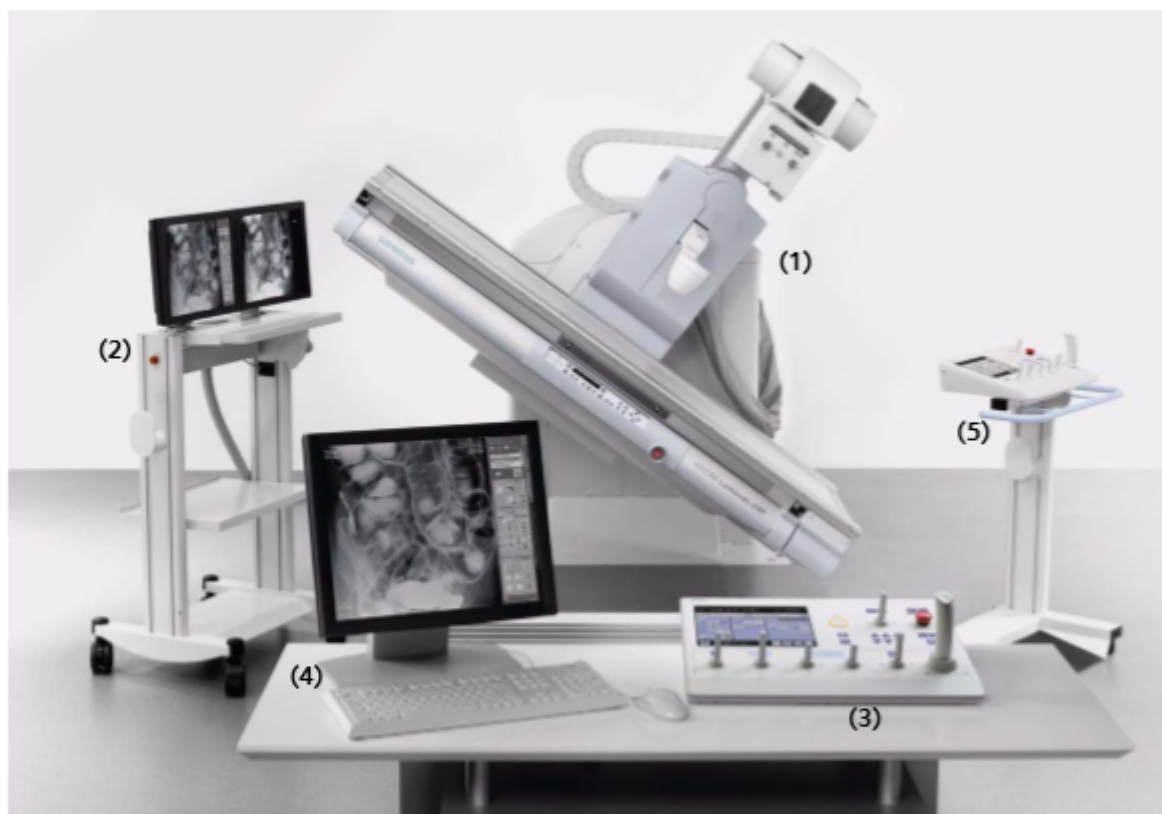
Luminos dRF Max

- (1) Luminos dRF examination unit
- (2) Monitor trolley
- (3) System remote control console
- (4) FLUOROSPOT Compact with monitor, keyboard, and mouse
- (5) Moveable tableside control trolley with foot switch for fluoroscopy and exposure

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
System remote control console¹

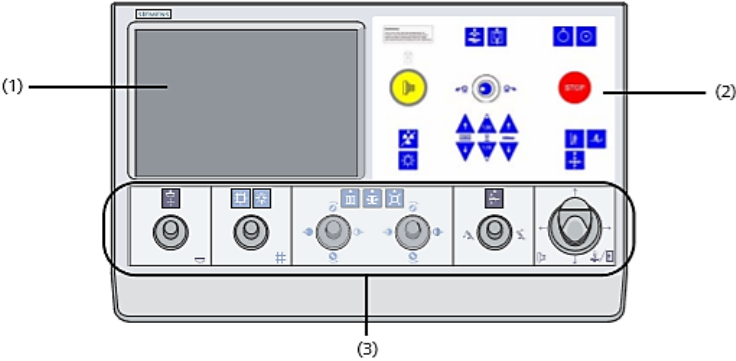
Touchscreen User Interface (TUI)

The system remote control console consists of three main parts:

- (1) Touchscreen User Interface TUI
- (2) Hardware buttons panel User Interface
- (3) Modular joystick User Interface²

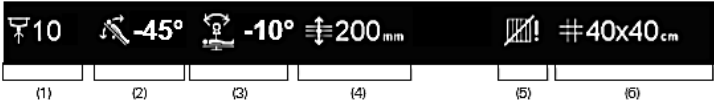Status area

- (1) Display of the set compression level (0 to 15), only in connection with the compression device
- (2) Table tilt angle
- (3) Oblique projection angle
- (4) Fulcrum height (axial oblique projection = isocenter) or tomographic height
- (5) Grid status
- (6) Collimated field of view

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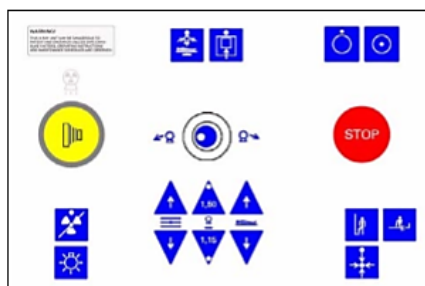
Hardware buttons panel user interface

"Radiation ON" indicator
– Symbol lights up orange

Exposure release button with pre- and main contact
– An acoustic signal sounds¹

Button with LED to block or unblock exposure release.

We recommend that this button is used at all times, except when a procedure is in progress. This is to prevent the possibility of radiation being emitted through the inadvertent actuation of an exposure release button.

Room light on or off (on-site installation required)

Moving the tube into 0° position

Enabling or disabling the tube detector coupling

Joystick for the oblique beam angulation

In the upper left corner is a blue LED for the activation state of the dead man's grip (DMG)
– LED is on: DMG is activ

- ☐ Fulcrum height for oblique projection (= isocenter)
- ☐ Tomographic height setting above tabletop

- ☐ Increasing SID up to 150 cm with indication when reached

- ☐ Decreasing SID up to 115 cm with indication when reached

Raising or lowering the table

ON button

OFF button

Red emergency STOP button

Automatic system positioning for upright patient transfer

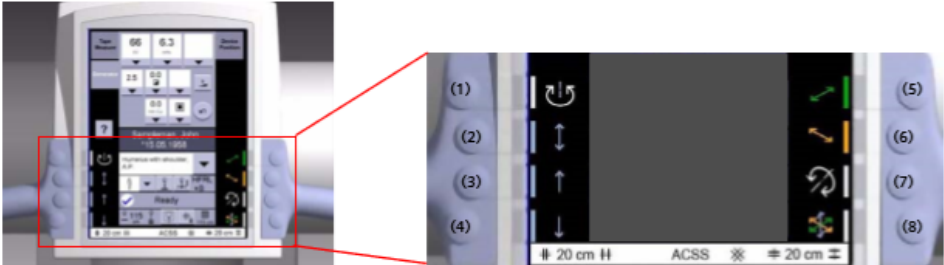
Automatic system positioning for horizontal patient transfer

SmartMove button with LED for starting movement control
Moves system into predefined OGP positions.

Kollimator - takstativ

Button number	Icon	Description
(1)		Tube rotation about the vertical axis
(2)		Vertical tube movement up and down
(3)		Vertical tube movement up (motorized)
(4)		Vertical tube movement down (motorized)
(5)		Longitudinal tube movement
		Centering display
(6)		Transverse tube movement
		Centering display
(7)		Tube rotation about the horizontal axis
(8)		Floating tube movement (longitudinal, transverse, vertical)

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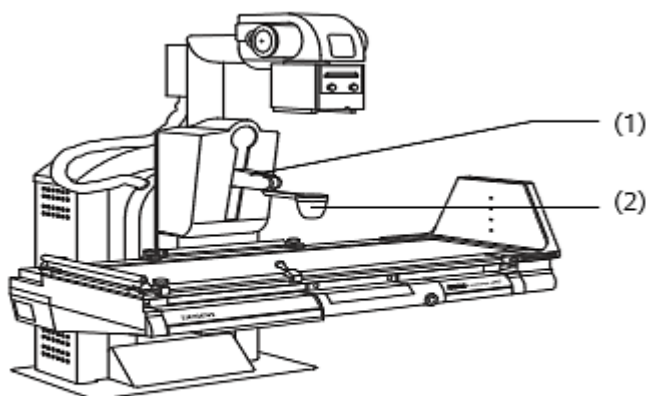
Kompresjonsskål:

- (1) Compression device
- (2) Compression cone

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